

KIT COMPONENTS FOR 2 TESTS PACK SIZE (DC-30-0017)

1 Quick Guide



1 Instruction for Use

2 Test Cassettes
in a Sealed Foil Pouches2 Sample
Collection Tubes

2 Collection Papers

STEP 1. COLLECT AND PREPARE STOOL SPECIMEN

1. Remove tape covers from back sides of collection paper.



2. Loosely place paper at the back portion of the toilet bowl, affix tape, and then lower the seat.




3. Deposit stool specimen on collection paper.



6. Insert the sampler with specimen back into the collection tube, firmly tighten, and shake the tube to mix the liquid. Flush the remaining stool and paper.




5. Using the grooved tip of sampler, pierce the stool in at least 5 different sites.

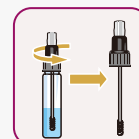


4. Unscrew and remove the sampler from the sample collection tube.

STEP 2. PERFORM THE TEST AND READ THE RESULT




1. Open sealed foil pouch, remove test cassette, and place on a flat surface.

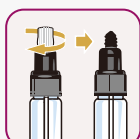

2. Holding the collection tube upright, unscrew the clear cap.

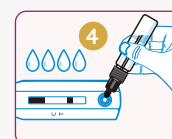

3. Squeeze the collection tube to deliver **four drops** of the sample into the sample well as shown.

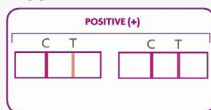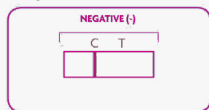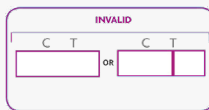POSITIVE:**NEGATIVE:****INVALID:**

5. Follow the reading instruction below to identify whether your result is positive, negative, or invalid. For positive results, consult a doctor.



4. Within 4-7 minutes, read the result window of the cassette.

NEED HELP?

Email order@diacarta.com or call 1-800-246-8878
9AM-5PM Pacific Standard Time, Monday-Friday

Distributed by DiaCarta Inc.
4385 Hopyard Rd, Suite 100
Pleasanton, CA 94588, USA
www.diacarta.com

