



DIACARTA™



MADE
IN THE **USA**

OTC

Are you at risk of Colorectal Cancer ?

Colorectal cancer (CRC) is the 2nd leading cause of cancer related deaths

DiaCarta's
SPOT ON
*Precision Diagnostics
brings you...*

iCOLON™ iFOB TEST - Immunochemical Fecal Occult Blood Test (Linear Flow)

- A simple Over The Counter (OTC) stool test
- Highly specific to human hemoglobin
- High sensitivity at 98%
- Easy to read results between 4-7 minutes
- No diet restrictions



Test configuration:

Over the Counter (OTC): 2 tests / pack Cat # DC-30-0017

iCOLON™ iFOB TEST **OTC**

Immunochemical Fecal Occult Blood Test

KIT COMPONENTS FOR 2 TESTS PACK SIZE (DC-30-0017)



1 Quick Guide



1 Instruction for Use



2 Test Cassettes
in a Sealed Foil Pouch



2 Sample
Collection Tubes



2 Collection Papers

STEP 1. COLLECT AND PREPARE STOOL SPECIMEN



1. Remove tape covers from
back sides of collection paper.



2. Loosely place paper at the back portion of the
toilet bowl, affix tape, and then lower the seat.



3. Deposit stool specimen on collection paper.



6. Insert the sampler with specimen back into
the collection tube, firmly tighten, and shake the
tube to mix the liquid. Flush the remaining stool
and paper.



5. Using the grooved tip of sampler, pierce the
stool in at least 5 different sites.




4. Unscrew and remove the sampler from the
sample collection tube.

STEP 2. PERFORM THE TEST AND READ THE RESULT



1. Open sealed foil pouch, remove test
cassette, and place on a flat surface.

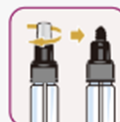


2. Holding the collection tube upright,
unscrew the clear cap.



3. Squeeze the collection tube to deliver one drop
of the sample into the sample well as shown.



5. Follow the reading instruction below to identify whether your result is positive, negative, or
invalid. For positive results, consult a doctor.

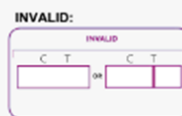

4. Within 4-7 minutes, read the result
window of the cassette.