


UTI-Detect Urine Collection and Shipping Instructions

PART 1. KIT COMPONENTS

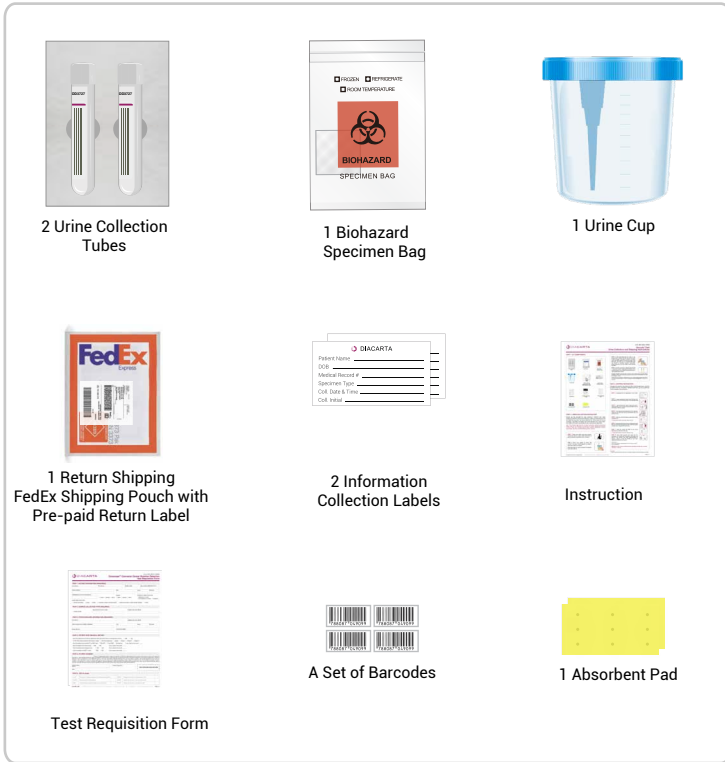

PART 2. URINE COLLECTION INSTRUCTION

Please use the provided BD Cups (Catalog # 364975) and Tubes (Catalog # 364951), Information Collection Labels, and Barcodes provided inside the UTI-Detect Test Urine Collection Kit. Other tubes will not be accepted. DiaCarta is not liable if the urine collection kit or urine collection tubes are found to be tampered upon receiving the specimen.

STEP 1. While over toilet, start urine stream. During mid-stream, place sterile specimen in path of stream and collect 50 mL of urine.



STEP 2. Place cup upright on clean, flat surface. Container may be tipped at an angle if specimen volume is limited. Peel back label on cap to expose the integrated transfer device.



STEP 3. Place evacuated tube into cavity on cap, stopper down. Advance the tube over puncture point to pierce stopper. Hold tube in position until filled (4mL per tube). Repeat to fill both urine tubes. Treat the screw cap of the cup as a contaminated sharp and discard in biohazard container approved for sharps disposal as per your facility's recommended procedure.



STEP 4. Confirm each tube is labeled with the date of collection and two unique patient identifiers such as patient date of birth, patient first and last name, or order identification number.



Important:

Preferably on the same day of collection, ship via clinical priority overnight delivery by shipping with FedEx shipping bag.

After urine collection, urine in the BD tube can be stored at room temperature for up to 48 hours including shipping.

PART 3. SHIPPING INSTRUCTION

Package and mail the specimen(s) to the DiaCarta laboratory. Each kit should be used for one patient.

STEP 1. Complete the Test Requisition Form (TRF).



STEP 2. Place one Barcode Label on the TRF, then fold and insert the TRF into the rear pouch of the biohazard bag.



STEP 3. Place the Barcode Labels on each tube with in vertical position. Don't overlap the barcode labels with the Information Collection Labels.



STEP 4. Place filled and barcode-labeled Urine Collection Tubes into the foam tray, remove the protective film and close the cover.



STEP 5. Place the filled foam tray into the Biohazard Bag, then put all biohazard bags that contain individual patient samples into FedEx shipping pouch.



STEP 6. Place FedEx pre-paid return label onto the FedEx shipping pouch. Record the kit tracking information and keep for your records. Call FedEx (800.463.3339) to request a pick-up or drop the package at your site's designated FedEx pick-up location.



Ship samples to:

DiaCarta Clinical Laboratory 4385 Hopyard Rd, Suite 100 Pleasanton, CA 94588

Phone: 1-800-246-8878

IMPORTANT: Ship samples to DiaCarta Monday-Thursday. DO NOT ship on Fridays, Saturdays, or Sundays.

REFERENCE

1. Clinical and Laboratory Standards Institute. GP41-Ed7, Collection of Diagnostic Venous Blood Specimens, 7th edition.